# Dr Brown's

natural flow®

BPA Free

## INSTRUCTION BOOKLET

### Manual Breast Pump + Feeding Bottle

English . . . . .	2	Français . . . . .	23	Русский . . . . .	45
Español . . . . .	7	Nederlands . . . . .	29	Ελληνικά . . . . .	51
Português . . . . .	12	Svenska . . . . .	35	Polski . . . . .	57
Italiano . . . . .	17	Norsk . . . . .	40		

## PARTS DIAGRAM

- A. Collection Bottle
- B. Bottle Adapter
- C. Valve
- D. Connector
- E. Breastcup
- F. Cap Insert
- G. Cap
- H. Handle

## DISASSEMBLY

- 1 Loosen and remove **Bottle** from **Adapter**.
- 2 Loosen and remove **Adapter** from **Connector**.
- 3 Remove the white **Valve** from underneath the **Connector**.
- 4 Remove **Breastcup** by gently pulling on the perimeter tabs.
- 5 Remove **Cap** assembly by pulling up on the **Cap** tab at the back of the pump.
- 6 Remove **Handle** by pushing the stem of the **Cap Insert** upward, then unhinging to separate.
- 7 Remove **Cap Insert** from **Cap** by pulling down.

## CLEANING

Before your first use and after all other uses, it is important to wash your pump.

Make sure all pump parts have been disassembled for cleaning.

All pump parts can be hand-washed or washed in a dishwasher.

**Note:** The white Valve and Breastcup are delicate. Please take extra care in cleaning them. It is recommended that you hand wash these parts. Do not use any rough brushes to clean these parts.

**TO HAND WASH:** Wash all parts in warm, soapy water, rinse well and air dry.  
**IN DISHWASHER:** All pump parts are top-rack dishwasher safe. Completely air dry parts and let them cool.

## ASSEMBLY

After cleaning, follow these steps to assemble your pump. Make sure all parts are completely dry from cleaning before any assembly.


Insert the white **Valve** to the underside of the **Connector**.

Screw on the **Bottle Adapter**. Tighten gently.

Screw the **Bottle** to the **Bottle Adapter**. Tighten gently.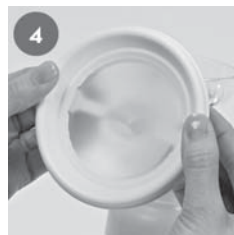

Snap the **Breastcup** on to the **Connector**. Press edges firmly to make sure it's secure all around.

Install **Cap Insert** on to the **Cap**, aligning the two pegs in the holes. Press the parts firmly together.


Attach **Handle** by pressing upward on the **Cap Insert** to extend the stem.

## TABLE OF CONTENTS:

Parts Diagram	2
Disassembly	3
Cleaning	3
Assembly	3-4
Expressing Breastmilk	4-5
Storing & Handling Breastmilk	5
Thawing & Preparing Breastmilk	6
Tips	6
Troubleshooting	6

Latch the hole in the **Handle** on to the **Stem** while sliding the **Handle** on to the two white handle pegs.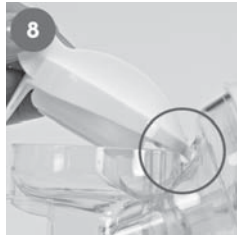

Hinge the front of the **Cap** assembly on to the clear hinge tab on the **Connector**.

Press the **Cap** assembly on to the **Connector** until you hear a snap.

## EXPRESSING BREASTMILK

### Flexible Breastcup

We designed this pump with a unique breastcup that is very soft and flexible. It will conform to and move with your breast as you express. Our mums have told us that the flexible breastcup makes for a more comfortable pumping session. There is no friction or tugging, which can cause trauma to your breast, especially over time.

### Getting Started

- 1 Make sure your hands and breasts are clean.

Position the **Breastcup** nipple centered, to your breast just firm enough to ensure a complete seal with no air leaks.

Begin pumping by squeezing the **Handle**.

Once you have Let-Down, continue to pump at a rate that is comfortable for you. Longer, slower squeezes often yield the most milk flow as these are similar to the actual nursing phase of breastfeeding.

Mums often achieve Let-Down with short, quick squeezes as these are similar to the fast, non-nutritive sucking motion of baby during the first phase of breastfeeding. This should initiate milk flow which is known as Let-Down. **Let-Down is the biological trigger to release breastmilk and it's the first step for a successful pumping session.**

Press side of breast to break suction.

Breastmilk should begin to flow and collect in the bottle. Even if you do not experience milk flow immediately, continue pumping for a few minutes. Your first few times expressing may not yield any milk. This is normal. Never exceed 5 minutes of pumping without milk flow.

Always clean your pump after every pumping session.

**Warning:** Pumping should not cause pain or discomfort. If you experience discomfort during pumping, contact a Lactation Consultant or your healthcare provider.

## STORING & HANDLING BREASTMILK

After expressing your breastmilk, it is important to store and handle it properly.

- ⦿ Make sure your hands and storage containers are clean to avoid contamination of your expressed breastmilk.
- ⦿ Use Breastmilk Collection Bottles or Breastmilk Storage Bags to store your breastmilk. These can be used in the refrigerator or freezer.
- ⦿ Be sure your storage bottle or bag is sealed tightly to ensure freshness of the breastmilk.
- ⦿ Date code your bottles or bags if you do not intend to use your breastmilk immediately.
- ⦿ For extended storage, freeze your breastmilk.

**Warning:** Do not add fresh breastmilk to already frozen breastmilk in a storage container. Do not save breastmilk from a used bottle for another feeding.

### Storage Duration of Fresh Breastmilk (for use with healthy full term infants)

LOCATION	TEMPERATURE	DURATION
Countertop, table	Room temperature (up to 25° C) <i>Containers should be covered and kept as cool as possible; covering the container with a cool towel may keep milk cooler.</i>	<b>6-8 hours</b>
Insulated cooler bag	-15-4° C <i>Keep ice packs in contact with milk containers at all times, limit opening cooler bag.</i>	<b>24 hours</b>
Refrigerator	4° C <i>Store milk in the back of the main body of the refrigerator.</i>	<b>5 days</b>
Freezer compartment of a refrigerator	-15° C	<b>2 weeks</b>
Freezer compartment of refrigerator with separate doors	-18° C	<b>3-6 months</b>
Chest or upright deep freezer	-20° C <i>Store milk toward the back of the freezer, where temperature is most constant. Milk stored for longer durations in the ranges listed is safe, but some of the lipids in the milk undergo degradation resulting in lower quality.</i>	<b>6-12 months</b>

Reference: Academy of Breastfeeding Medicine. (2004) Clinical Protocol #8: Human Milk Storage Information for Home Use for Healthy Full Term Infants. Princeton Junction, New Jersey: Academy of Breastfeeding Medicine.

## THAWING & PREPARING BREASTMILK

After storing your breastmilk, follow these steps to prepare it for baby:

- ⦿ To thaw breastmilk, place the sealed Collection Bottle or Storage Bag in refrigerator or run under cool tap water.
- ⦿ To warm breastmilk, place the sealed Collection Bottle or Storage Bag in warm tap water.
- ⦿ Always check breastmilk temperature before feeding baby.
- ⦿ Use thawed breastmilk immediately or refrigerate up to 24 hours.

**Warning:** Do not microwave breastmilk. Do not microwave Collection Bottles or Storage Bags containing breastmilk. Microwave ovens can create uneven heating which could scald baby and deteriorate the nutrients of breastmilk.

**Note:** Do not thaw breastmilk in very hot or boiling water. Once thawed, do not refreeze or refrigerate breastmilk again. Do not allow thawed breastmilk to sit at room temperature for more than 2 hours.

## TIPS

Breastfeeding and pumping are unique experiences for nursing mums. Please review the tips below before pumping.

- ⦿ Mums are more likely to successfully pump if they are relaxed and thinking about baby. Avoid pumping if you are rushed.
- ⦿ Efficient pumping usually takes patience and practice for mums. Mums that pump frequently become accustomed to it and are more likely to have faster Let-Down and more milk flow.
- ⦿ All mums are different. Some experience Let-Down quickly and some mums do not.
- ⦿ Some express on their first try and some mums do not.
- ⦿ Do not be discouraged if you are unable to express milk on your first try. Simply take a break and try again later.
- ⦿ Do not be discouraged if you express only a small amount of breastmilk. This is common especially on the first few attempts.
- ⦿ Pumping consistently, in addition to breastfeeding, can help sustain your milk supply over time.
- ⦿ A warm compress or a warm bath/shower can help encourage milk flow.
- ⦿ Most important of all, pumping should be a comfortable experience. If you feel pain or discomfort during pumping, contact a Lactation Consultant or your healthcare provider.

## TROUBLESHOOTING

- ⦿ For your pump to work as designed, all parts must be properly assembled. Make sure all parts are snugly fit so you have an air tight pump.
- ⦿ When applying to your breast, make sure the Breastcup is centered around your nipple. Hold the pump firm enough to your breast for a complete seal all around, but not too firm as this could inhibit milk flow.
- ⦿ When the Handle is squeezed, make sure that you feel a light suction on your breast.
- ⦿ If you require personal assistance about your pump, or have lost or broken parts, please contact your local distributor.

**THIS IS A SINGLE USER PRODUCT THAT SHOULD NOT BE SHARED BETWEEN MUMS.**

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©2014 HANDI-CRAFT™ COMPANY
Stilotex S.A.S
Careb Trading AB
Exima Trading LLC